

**DESIGN, MANUFACTURING , MARKETING & QUALITY CONTROL OF MEDICAL DEVICES**  
Via Faentina 207/Q 48124 Fornace Zarattini (RA) - **ITALY** TEL. +39-0544-464048 FAX +39-0544-501617  
E-mail: [commerciale@vasinistrumenti.it](mailto:commerciale@vasinistrumenti.it) Website: [www.vasinistrumenti.it](http://www.vasinistrumenti.it)

**ROLLER PLIER FOR PVC TUBES STRIPPING WITH  
CUTTER MOD. MTS-01  
WITHOUT CUTTER MOD. MTS-02**



**MTS-01  
WITH CUTTER  
CND: B99  
RDM: 2213281**

**MTS-02  
WITHOUT CUTTER  
CND: B99  
RDM: 2213354**

## USE INSTRUCTIONS


### INTENDED USE

The manual multifunction roller plier is a manual medical device used for blood bags' tubes stripping and cutting.

### GENERAL FEATURES

Product Codce	MTS-01
Body Material	Anodized aluminum
Roller Material	Delrin
Weight	85 gr
Dimensions	170 x 73 x 25h mm
Quantity	1 piece/box
Medical Device	Class I

## NAMES

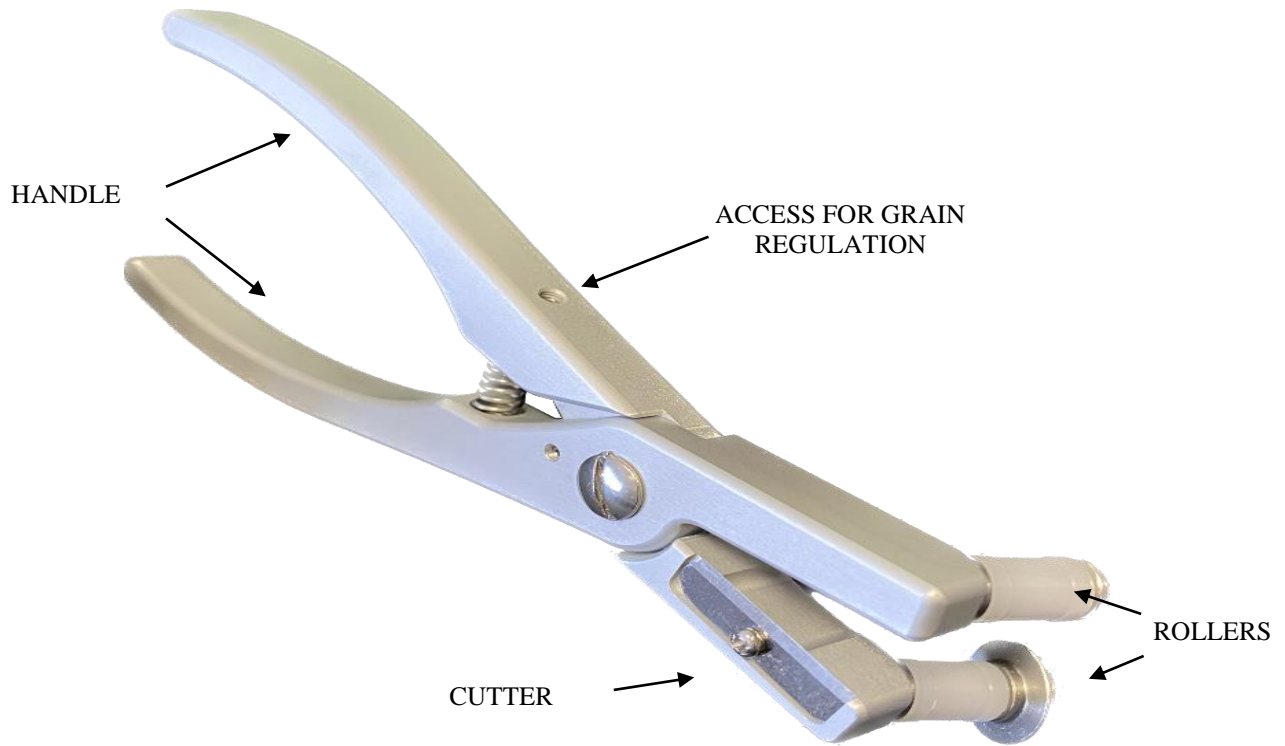


## FUNCTIONS

### 1. STRIPPING


From the needle extremity, shrink toward the blood bag by putting the tube between the two rollers and pulling the plier.

With your hand, exercise a light pressure on the plier's handle so that you can gently push the remaining blood into the bag.

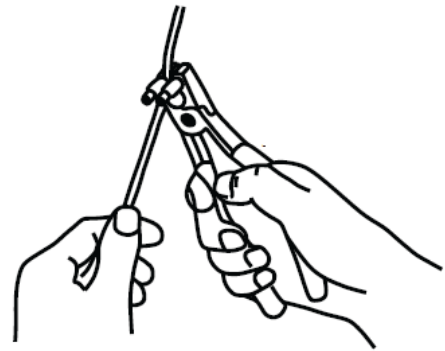


### 2. SEALING

Put a sealing clip on two points of the folded tube or seal it with a radio frequency tubes sealer.

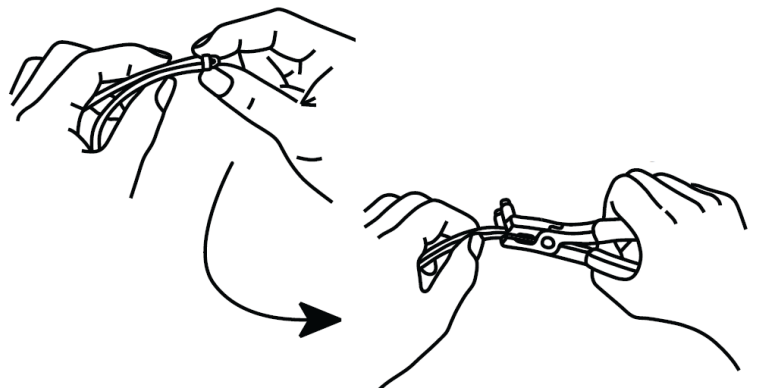

### 3. CUTTING

Cut the tube between the two seals by putting the tube in the proximity of the cutter and push the plier's handle. This is only valid for mod. MST-01

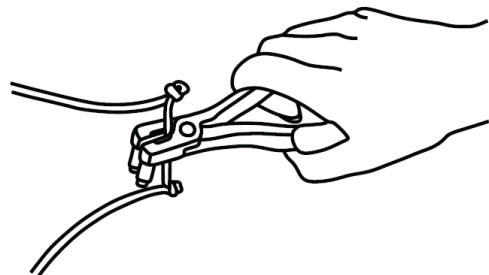







The external spring system has an internal grain with hexagon Socket that regulates the final closure of the rollers and keeps the handles in the open position, facilitating the use of the gripper by the operator. Acting on the grain with hexagon socket with a special key, the minimum distance of the rollers is adjusted during the stripping phase.

## HOW TO CLEAN IT

Clean the device with a soft cloth soaked in isopropyl alcohol. Then wipe with a soft, dry cloth. Do not use corrosive or abrasive cleaners.

## SYMBOLS AND DEFINITIONS

Symbols	Description
 VASINI STRUMENTI S.r.l. Via Faentina 207/Q Fornace Zarattini (RA) e-mail info@vasinistrumenti.it	Medical Device Manufacturer
	<b>MODEL/TYPE</b> The model identifies the product's family, the type identifies the device inside the family.
	Device's serial number
	Production date
	CE Marking
	Medical Device

## WARRANTY

The manufacturer guarantees the device and all its components for 12 months since the delivery date. During this period, Vasini Instruments srl undertakes the responsibility to repair or replace the device for free when there are manufacturing defects, only if the found defect is communicated in advance. Shipment costs that come with goods repair are at the client's charge, both in case of goods delivery and pick-up. Replaced parts remain a Vasini Instruments property. Warranty can not be considered valid when the followings occur: tampering with, modification, inappropriate use with respect to the use instructions, wrong power supply, repairs done by unauthorized personnel, accidental breakages due to transport or falls; lack, cancellation or alteration of the serial number. In case of repairs not covered by warranty, Vasini Instruments Srl or one of its representatives will draw up an estimate and will arrange for the repair with the prior written consent of the customer. Each return must be authorized in advance by Vasini Instruments Srl. In any case, the warranty excludes any compensation for damages.



**DESIGN, MANUFACTURING , MARKETING OF MEDICAL DEVICE FOR BLOOD TRANSFUSION CENTRES**

Via Faentina 207/Q 48124 Ravenna - **ITALY** TEL. +39-0544-464048 FAX +39-0544-501617

E-mail: [commerciale@vasinistrumenti.it](mailto:commerciale@vasinistrumenti.it) Website: [www.vasinistrumenti.it](http://www.vasinistrumenti.it)

# DECLARATION OF CONFORMITY



**The manufacturer:**

***Vasini Strumenti Srl***

Via Faentina 207/Q

48124 FORNACE ZARATTINI (RA) - Italy

SRN: **IT-MF-000012098**

**DECLARES under his full responsibility**

that the medical device defined as

**MANUAL STRIPPER FOR PVC TUBES**

MANUAL STRIPPER FOR PVC TUBES WITH CUTTER	<b>MTS-01</b>
MANUAL STRIPPER FOR PVC TUBES WITHOUT CUTTER	<b>MTS-02</b>

**BASE UDI: 8057502560000PINZMX**

It is a Class I non-invasive medical device as per rule 1, encl. VIII of MDR 2017/745.

This device complies with the essential safety requirements indicated in Annex I and follows the prescription of MDR2017 / 745.

Certification procedure as per Annex II and Annex III of MDR2017 / 745.

Ravenna (RA) - Italy, 18/02/2022

In faith

The Legal Representative

Marco Vasini